


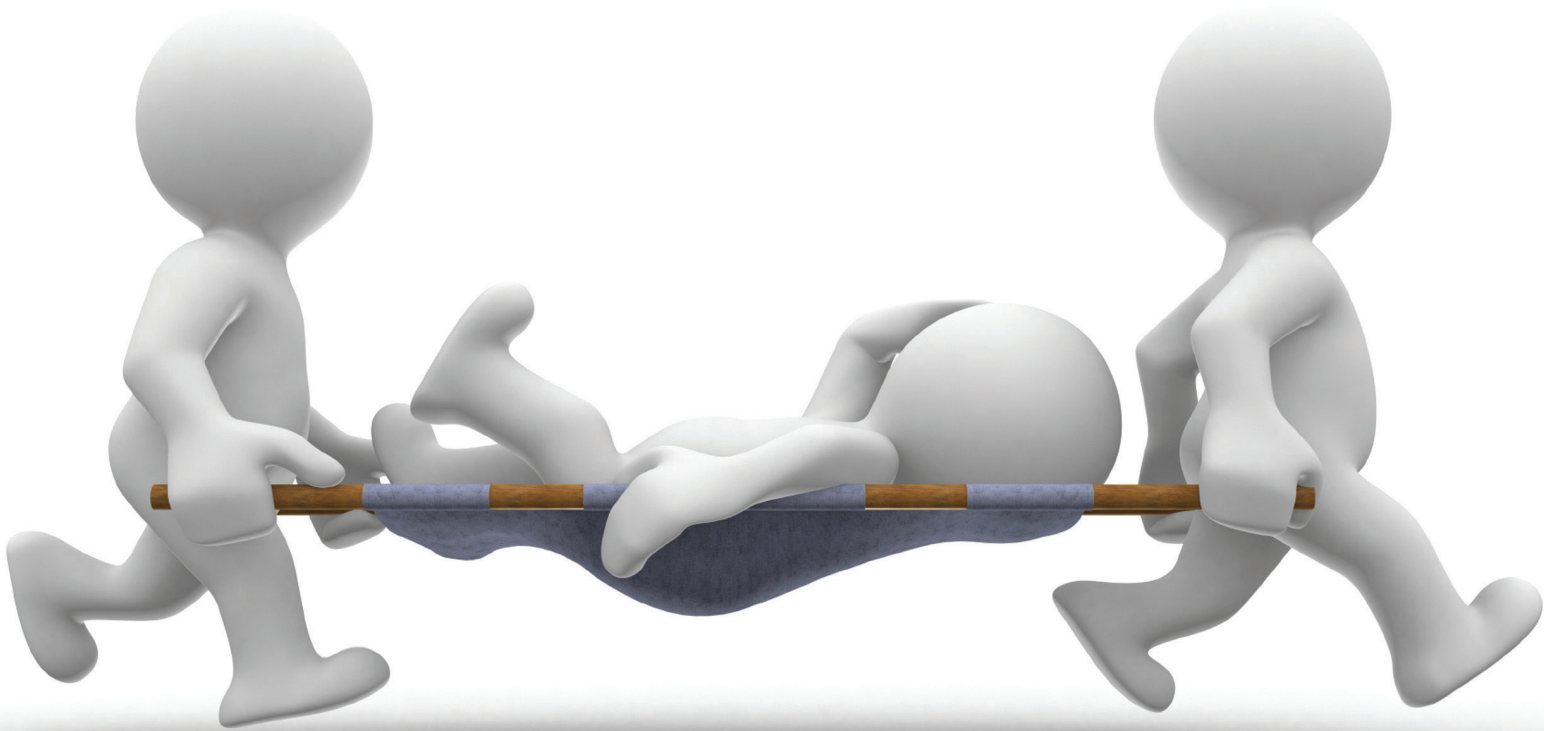

“Emergency”




Life Control[®] ReSt


Disposable Mechanical Resuscitator

Features

- ▶ Disposable.
- ▶ 50% -100% oxygen concentration can be selected with the FiO₂ Mixer.
- ▶ Safely used in MRI and all magnetic fields.
- ▶ Body fixation feature (Profix).
- ▶ Compatible with all lung structures.
- ▶ Complies with ISO 10651-5 Standard.
- ▶ No risk of Barotrauma.
- ▶ Applied by users trained in CPR (Cardiopulmonary Resuscitation).
- ▶ Should be used in patients over 10 kg.
- ▶ Cardiac massage can be performed during ventilation.
- ▶ Light and safe, PEEP sensitive.

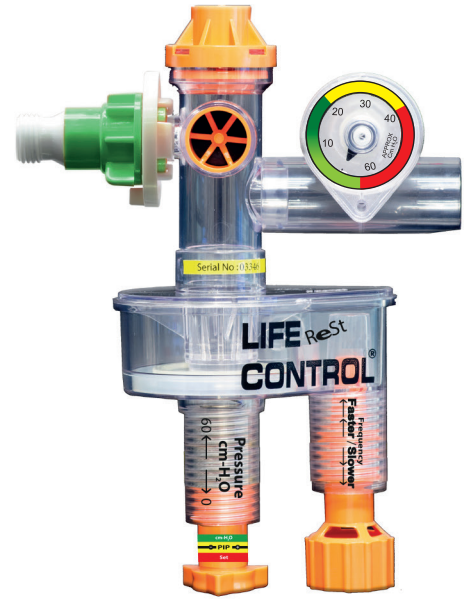


Connections

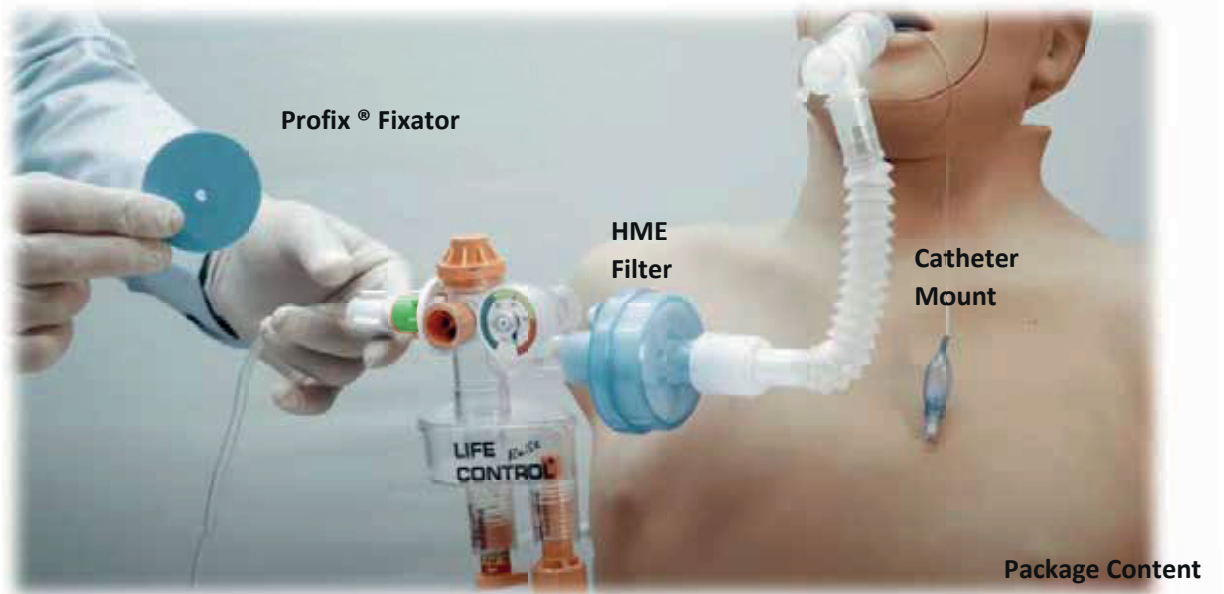

Usage Areas

- ▶ Air, Land and Sea Ambulances,
- ▶ Emergency Units,
- ▶ Intensive Care Units,
- ▶ Anesthesia Units,
- ▶ In-hospital and out-of-hospital patient transport,
- ▶ Military Units,
- ▶ Hotels and Holiday Villages,
- ▶ Open Area Conditions

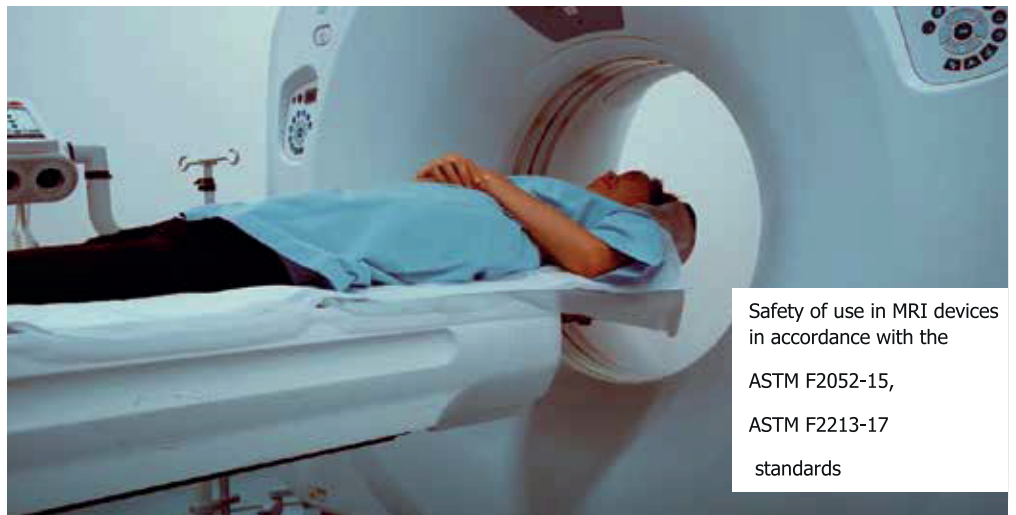


Safety of use in MRI devices in accordance with the ASTM F2052-15, ASTM F2213-17 standards

Life Control[®] ReSt

Disposable Mechanical Resuscitator

Functions

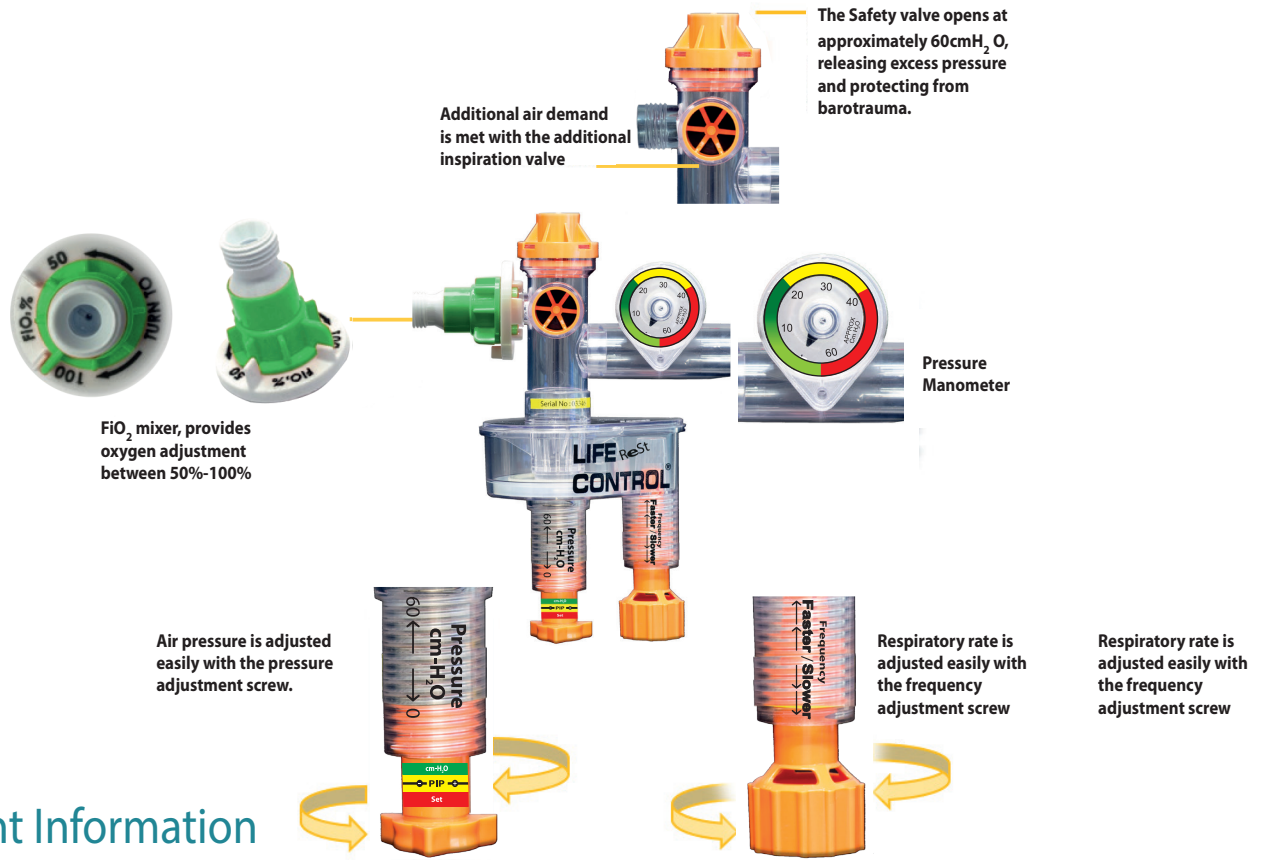

Important Information

- ▶ Life Control[®] should be used in emergency services, intensive care, field and normal use" UNDER THE CONTINUOUS SUPERVISION OF QUALIFIED PERSONS" in accordance with the ISO 10651-5 standart
- ▶ Electrical supply (battery, accumulator.. etc) is not required to operate Life Control[®]. Pressurized oxygen supply is sufficient for operation.
- ▶ Life Control[®] works effectively with air sources that provide a flow in range of 8-40 L / min and a pressure value in the range of 2.5- 5.5 bar.
- ▶ HME filter has 99,997 retention and it is disposable.
- ▶ If daege occurs to the filter due to vomit or secretion, the filter should be removed and replaced within maximum 20 seconds.




Model

| Reference Code | DMO Code* | Product Description | Product Description |
|----------------|-----------|--|---------------------|
| MREST | SMT1377 | Disposable Mechanical Resuscitator with HME filter | Life Control Rest |

* It applies to Turkey. It is the Health Market Code.

**The design and design modification rights of all components of the life control device are reserved by TMT Tibbi Medikal Malz. San. Ve Tic. Aş company.

Egemen around the world!

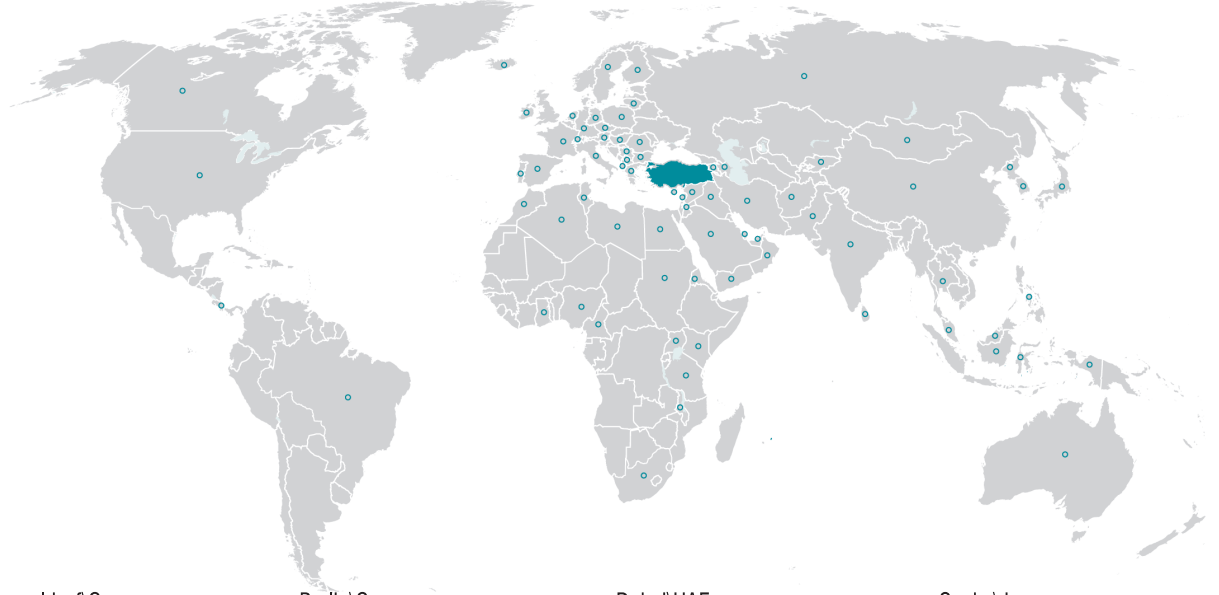

Dusseldorf\Germany




Berlin\Germany



Dubai\UAE



Osaka\Japan

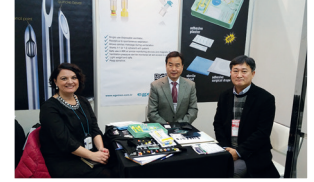


Hong Kong\China



Cape Town\Africa




London\UK

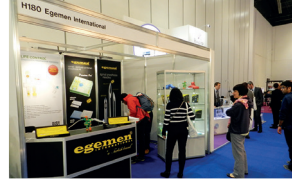


Sao Paulo\Brazil



Maastricht\Netherlands

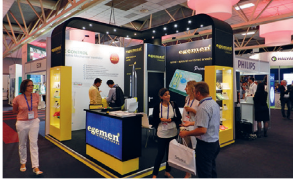

Ljubljana\Slovenia

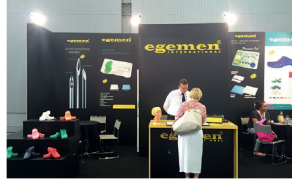

Seoul\S Korea



Erbil\Iraq



TMT TIBBI MEDİKAL MALZEMELERİ SAN. VE TİC. A.Ş.

Fatih Mahallesi 1188 Sokak No: 14 Sarnıç - Gaziemir
İZMİR TURKEY

Phone: +90 232 259 32 10

Fax: +90 232 277 47 15

info@egemen.com.tr www.egemen.com.tr

KGF-731004-A Rev.Date/No: 26.05.2023./4



egemen.international



EGEMEN INTERNATIONAL



Egemen International



@Egemeninternati